



SafeTouch[®] II

SAFETY FISTULA NEEDLE



Features and Benefits of SafeTouch II

- Built-in safety advantage: a permanent locking feature eliminates the worry of accidental needlesticks
- Needle retracts into the safety guard
- Textured wings provide a secure grip, making it easier to cannulate




**NIPRO CONNECT[®]
PROGRAM**

Our commitment to strengthen every partnership.

All Nipro renal products are backed by the Nipro Connect Educational Program – offering clinicians product evaluations, in-services, continuing education units (CEU's) and instructional materials.


www.nipro.com

SafeTouch II Needle Removal Technique

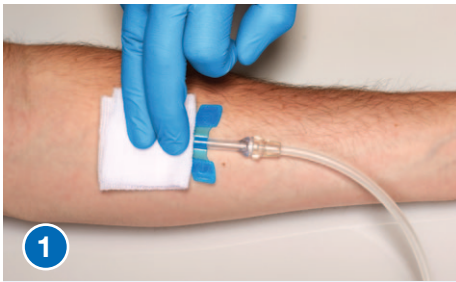


1

Remove all securing tapes from the wings and the access area. Place a sterile dressing over the needle access site.

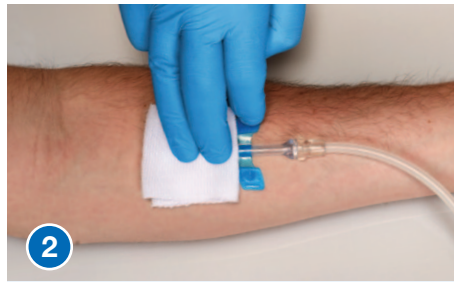

2

Maintain gentle pressure on the needle access site, while holding down the wing with one finger.

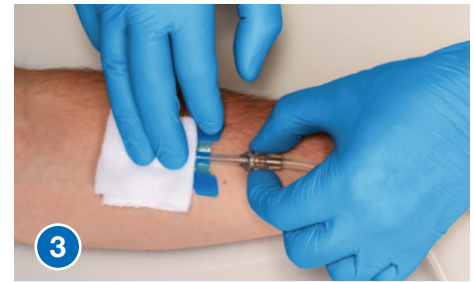

3

Release the lock by pressing both sides of the stopper. Slide the stopper back to retract the needle until you hear an audible click.



4

The safety guard is now fully engaged. Dispose in approved biohazard sharps container.

ORDERING INFORMATION

Item #	Description	Packaging	Item #	Description	Packaging	
FS+142530BC	14GX1 Fixed Hub 12" Tubing	50/Box 10 Boxes/Case (500)	FS+163230BCR	16GX1.1/4 Rotating Hub 12" Tubing	50/Box 10 Boxes/Case (500)	
FS+152530BC	15GX1 Fixed Hub 12" Tubing		FS+172530BCR	17GX1 Rotating Hub 12" Tubing		
FS+153230BC	15GX1.1/4 Fixed Hub 12" Tubing		FS+173230BCR	17GX1.1/4 Rotating Hub 12" Tubing		
FS+162530BC	16GX1 Fixed Hub 12" Tubing		FS+152530CR	15GX1 Rotating Hub w/o Backeye 12" Tubing		
FS+163230BC	16GX1.1/4 Fixed Hub 12" Tubing		FS+162530CR	16GX1 Rotating Hub w/o Backeye 12" Tubing		
FS+172530BC	17GX1 Fixed Hub 12" Tubing		FS+172530CR	17GX1 Rotating Hub w/o Backeye 12" Tubing		
FS+173230BC	17GX1.1/4 Fixed Hub 12" Tubing		FS+142530TP	14GX1 Twin Pack 12" Tubing		2/Package Red – Arterial Blue – Venous
FS+173230BC-G	17GX1.1/4 Fixed Hub 12" Tubing - Gamma		FS+152530TP	15GX1 Twin Pack 12" Tubing		
FS+142530BCR	14GX1 Rotating Hub 12" Tubing		FS+162530TP	16GX1 Twin Pack 12" Tubing		
FS+152530BCR	15GX1 Rotating Hub 12" Tubing		FS+172530TP	17GX1 Twin Pack 12" Tubing		25 Pairs/Box 10 Boxes/Case (250 Pairs)
FS+153230BCR	15GX1.1/4 Rotating Hub 12" Tubing	FS+152530TPR	15GX1 Twin Pack Rotating Hub 12" Tubing			
FS+162530BCR	16GX1 Rotating Hub 12" Tubing	FS+162530TPR	16GX1 Twin Pack Rotating Hub 12" Tubing			
			FS+172530TPR	17GX1 Twin Pack Rotating Hub 12" Tubing		

CAUTION: Federal law restricts these devices to sale by or on the order of a physician.



NIPRO MEDICAL CORPORATION
200 Crossing Blvd., Bridgewater, NJ 08807
T: +1 (908) 393-7030, F: +1 (908) 393-7031
www.nipro.com

© 2014 Nipro Medical Corp. All Rights Reserved.
SafeTouch is a registered trademark of Nipro Medical Corporation.
Printed in U.S.A. RN-DSUS-STII Rev 0